

Checklist CloudCuddle Junior daily check

Purpose

This checklist must be used by parents, caregivers and family caregivers during their daily check of the CloudCuddle Junior. The user's safety is guaranteed by carefully and thoroughly checking the items on this checklist.

Important

This daily check applies only as long as the CloudCuddle Junior remains assembled and used without any modifications. In the event of any change to the setup, relocation, or reassembly of the device, the user manual must be consulted and the complete post-assembly inspection, as described in the user manual, must be carried out.

If any inspection point does not meet the specified requirements, the user manual must be consulted for the required follow-up actions. For additional support, please contact CloudCuddle.

Inspection points before leaving the user unattended in the CloudCuddle Junior.

Frame

- ✓ The frame is fully inflated and can not be squeezed
- ✓ The two tube clamps at the top of the frame are closed
- ✓ The valve is properly closed
 - o Rear nut must be securely tightened
 - o The cord is not trapped
 - o Red valve cap must be securely tightened
- ✓ No visible damage to the frame
- ✓ No pressure loss visible or audible

Cover, mesh and sheet

- ✓ The cover is without damage
- ✓ The sheet and mesh are visibly taut and without damage
- ✓ There are no objects or loose materials in the CloudCuddle Maxx
- ✓ The fabric displays no signs of wear, holes or thinning

Fastenings and safety

- ✓ The CloudCuddle Maxx has been correctly secured to the bed

Zips and access ports

- ✓ All zips of the two access panels open smoothly, can be properly closed, and are in good condition
- ✓ The zip pullers of the two access panels are laid flat so they cannot be opened