

Checklist CloudCuddle Maxx daily check

Purpose

This checklist must be used by care providers during their daily check of the CloudCuddle Maxx. The user's safety is guaranteed by carefully and thoroughly checking the items on this checklist.

Important

You must only perform this daily check if no changes were made since the CloudCuddle Maxx was assembled and used. Whenever you make changes to the set-up of the device or if the device is moved or reassembled, you must always check the instructions for use. You must also perform a full check after reassembly, as described in section 6.1.1. of the instructions for use.

If an item on this checklist does not meet the specified requirements, please consult the instructions for use for the necessary follow-up steps. Contact Human Protection for additional support.

Items to be checked before you leave the user in the CloudCuddle Maxx unattended

Frame

- ✓ The frame is fully inflated and can not be squeezed
- ✓ The three tube clamps at the top of the frame are closed
- ✓ The valve is properly closed
 - o Rear nut must be securely tightened
 - o The cord is not trapped
 - o Red valve cap must be securely tightened
- ✓ No visible damage to the frame
- ✓ No pressure loss visible or audible

Cover, mesh and sheet

- ✓ The cover is without damage
- ✓ The sheet and mesh are visibly taut and without damage
- ✓ There are no objects or loose materials in the CloudCuddle Maxx
- ✓ The fabric displays no signs of wear, holes or thinning

Fastenings and safety

- ✓ The CloudCuddle Maxx has been correctly secured to the bed base and moves with the care bed when it is raised or lowered
- ✓ All buckles are fastened, work properly, and are without damage
- ✓ All tensioning straps are securely fastened
- ✓ The accessibility set is used in accordance with agreements made (if applicable)
- ✓ The bed brakes of the bed are engaged (if applicable)
- ✓ The locking function of the adjustable section has been activated and the manual control is blocked and not within reach of the person in the bed
- ✓ The bed has been lowered to its lowest position (max. 50 cm), except when care is administered

Zips and access ports

- ✓ All zips of the four access panels open smoothly, can be properly closed, and are in good condition
- ✓ The three medical access ports are closed (unless they are in use)
- ✓ The zip pullers of all three medical access ports are laid flat so they cannot be opened